



REUSABLE FACE SHIELD


PRODUCT INFORMATION & INSTRUCTIONS

SHEFFSHIELD is a PPE device designed to protect against airborne liquid droplets of substances and mixtures of harmful biological agents which are hazardous to health and cause illness. This device provides full face and lateral protection, featuring a liquid-proof, transparent visor to provide protection to the user.

ASSEMBLY

To assemble:

1. **Ensure** that the visor 'L' stamp is positioned on the user's left side when assembling so that the anti-fog coating is on the correct side;
2. **Push** the two front headband mounting studs through the middle two visor holes;
3. **Stretch** the headband arms outwards slightly and push the side headband mounting studs through outer two visor holes;
4. **Attach** elastic strap to the rear notch on each arm of the headband.

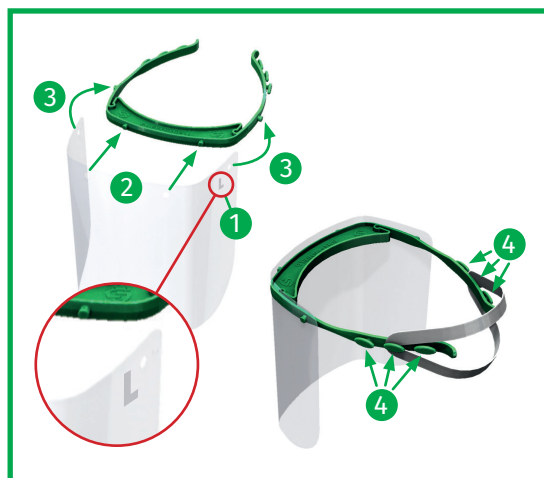

FITTING


To fit:

1. **Hold** headband arm ends;
2. **Place** elastic at the back of the head;
3. **Pull** the headband forward into position;
4. **Ensure** the elastic strap is at a comfortable position. If not, reposition elastic strap to a different notch.

REMOVAL

To remove:

1. **Hold** headband arm ends;
2. **Pull** the headband forward then up from the front of the face, releasing the elastic.

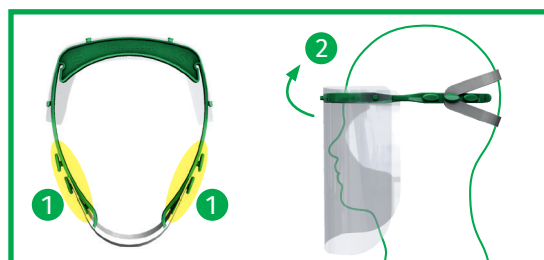

CLEANING & DISINFECTION

The face shield **can be cleaned and disinfected** using sodium hypochlorite, ethyl alcohol or isopropyl alcohol following suitable disinfection directions.

NOTES:

- › This protective face shield is manufactured for **COVID-protection only** – it is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.
- › The face shield may be used in conjunction with a respirator mask and/or additional eye wear such as glasses.
- › This product is BSI Certified to PPE Regulation for Healthcare Professionals 2020/403.



REUSABLE FACE SHIELD

PRODUCT INFORMATION & INSTRUCTIONS

MAINTENANCE

Regular inspection of the face shield should be completed and if any defects found, the item **discarded**:

1. **Ensure** the visor surface is **free from scratches** which could dangerously impair vision.
2. **Ensure** the headband and retaining strap are damage and wear free.

The face shield has a shelf-life of 3 years after manufacture and after this date should be **retired and recycled**.

STORAGE & TRANSPORT

The face shield should be stored in a **dry location** away from sources of heat and strong sunlight.

Do not store with any objects on top of the face shield, or by stacking the shield such that the head-band may become permanently deformed.

NOTES:

- › It is the responsibility of individual users to determine allergies to any materials used to construct this face shield; they should not use the product if allergic to the following:

HEADBAND - POLYPROPYLENE

VISOR - PET

STRAP - SILICONE RUBBER



**PLEASE RECYCLE WHEN
FINISHED WITH EACH PART.**



Certification: BSI's PPE for Healthcare Professionals 2020/403—Eye protection Technical Specification
Notified Body: BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP, UK
Notified Body No.: 0086
BSI Certificate No.: 729058

Manufactured by:
SHEFFSHIELD
AMP Technology Centre,
Brunel Way,
Rotherham,
S60 5WG, UK

For more information:
E: info@sheffshield.co.uk
T: 0114 3994757

Follow us online:
 SheffShield
 @SheffShield

sheffshield.co.uk